# Swabs for transport of microbiological samples

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 **deltalab**  
member of **SCGP**

DELTALAB is a leading company in design and manufacturing of single use Labware. Wherever DELTALAB supplies, it does so with commitment to Quality, Service and Business Ethics and Corporate Social Responsibility.

DELTALAB, with more than 50 years of experience, offers a diverse and market-adapted range of products.

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## Sterile swabs in round tube / Escobillones estériles en tubo

class IIa **MD** **CE** 0318

Dry swabs supplied in shockproof round bottom polypropylene tube, with a label sealing the cap. The tube (Ø13 x 165 mm) is labeled indicating code, description, lot number, expiry date, UDI, and providing an identifying area to note down collection details (patient or site, date,...). Moreover, the label seals the tube with the cap of the swab, acting like a tamper evident system.

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They are certified Human DNA, DNase and RNase free. Sterile.

Its components are a sampling swab packaged in a polypropylene tube, which protects the sample while it is being transported to the laboratory prior to its analysis. The stick of the swab is made of polystyrene while the head is produced with viscose. The tube is labeled indicating code, description, lot number, expiry date, bar code, and providing an identifying area to note down collection details (site, date,...). Moreover, the label seals the tube with the cap of the swab, acting like a tamper evident system.

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#### ✓ HIGH RECOVERY / ELEVADA RECUPERACIÓN

DELTALAB transport swabs allow the recovery of **Aerobic, Facultative Anaerobic and Anaerobic** microorganisms up to 48 hours, and 24 hour for Fastidious Bacteria.

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#### ✓ LOW BIOBURDEN / BAJO BIOBURDEN

DELTALAB only uses **high quality raw material** and all the manufacturing processes are performed in a **Clean Room** thus ensuring a very low bioburden level. Furthermore, the swabs are also subjected to a **terminal sterilisation process assuring total sterility**.

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#### ✓ TOTAL TRACEABILITY / TOTAL TRAZABILIDAD

Peel-pack indicating; **code, description, lot number, expiry date and UDI**. Every tube also carries a label specifying code, description, lot and expiry date.

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#### ✓ LONG-LASTING SHELF LIFE / LARGA VIDA ÚTIL

**30 months** of shelf life for bacteria media (Amies, Stuart and Cary Blair) and **12 months** for Chlamydia medium.

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## Amies

AMIES media is a modification of Cary Blair media, which is at the same time a modification of Stuart media. It assures viability of organisms such as:

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### AMIES CHARCOAL / CARBÓN

The charcoal improves the recovery ratio of fastidious bacteria, like *Neisseria gonorrhoeae* and *Vibrio cholerae*.

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## Stuart

The modified Stuart media allows the preservation and transportation of a large number of pathological microorganisms.

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## Cary Blair

Cary Blair is a modification of Stuart media. It is specially formulated for faecal collection and anaerobic transport.

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## Chlamydia

Liquid media suitable for Chlamydia. For cervical samples it is recommended to firstly use a dry swab to clear the cervical channel. Suitable for direct extensions on slides.

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