

en

NYLON FLOCKED SWAB

Intended use:

Sterile surgical-type invasive sample collection system by direct contact with the patient.

Intended for single use. The sterilisation method is shown in the primary packaging. Swabs are medical devices (MD) and are covered by CE0318.

Presentation:

The swab is composed of a polystyrene stick and a flocked nylon head. They have different shapes and sizes depending on the location of the sample collection: standard flocked swab, urethral flocked swab, nasopharyngeal flocked swab and minitip flocked swab. Product intended to collect and transport biological samples for microbiological analysis by professional health specialists and in appropriate hygienic conditions.

The flocked swabs are individually packed in a tube, peel-pack or flowpack.

The swab is intended to be used in two phases:

- 1) taking the sample by a healthcare professional in a clinical-hospital environment and transporting it.
- 2) processing of the sample by specialized health personnel in clinical-hospital settings. The use of the swab does not present difficulties for the intended users.

Precautions:

Not suitable for any other application than its intended use. Do not use if the packaging has any signs of breakage or damage. If you see that the unit package has lost its integrity, do not use the swab as there is no guarantee of its sterility. Do not use if you notice that the swab is dirty. Discard the product if you find any broken brushes, with edges or without a head. Only for single use, the reuse of this product may cause infections. Handle carefully. Any strong pressure during use, although unlikely, could break the swab. In the case of any damage, discard immediately. The swabs do not require any special storage conditions. After use, the swabs must be decontaminated or treated as contaminated medical waste. It is the responsibility of each laboratory to handle, treat and dispose the waste according to the current legislation. Unused swabs can be considered non-hazardous products and can be disposed of accordingly. Do not use if the expiration date has been exceeded. Do not use if any component is missing or if the printed product information is not displayed correctly.

Symbol glossary:

 Catalogue number	 Batch number	 Consult instructions for use on the website www.deltalabgroup.com/eifus	 Quantity	 Sterile by irradiation
 Keep dry	 Medical Device	 Do not use if package is damaged	 Do not re-use	 Sterile by Ethylene Oxide
 Keep away from sunlight	 Manufacturer	 Use-by-date	 CE marking	
 Unique device identifier	 Manufacturing date	 Unique sterile barrier system		

In case of a serious incident* related to the product, notify to Deltalab, S.L. as well as the competent authority of the State in which the user is established. *A "serious incident" is understood as one that entails the death, or serious deterioration of the health of the patient or user or a serious threat to public health.



DELTALAB, S.L.
 Plaza de La Verneda 1, Pol. Ind. La Llana,
 08191 Rubí, Barcelona. España/Spain.
info@deltalabgroup.com - www.deltalabgroup.com

  0318

Reference	Description
300262	SWAB IN TUBE PS FLOCK NASOPH.80MM
300264	SWAB IN TUBE PS FLOCK STANDARD 100MM
310206K	FLOCKED SWAB STANDARD 80MM FLOW
310207K	FLOCKED SWAB NASOPH. BP 80MM FLOW