



en

BED PAN AND URINE COLLECTION BOTTLE

For the collection of human biological fluids (urine and feces). Clean and disinfect before use. No sterilised.

The product is not manufactured with natural rubber latex or dried natural rubber.

Medical device.

Intended use:

Both the bedpan and the male urinal are intended for the collection of urine and/or faeces in bedridden patients or those with reduced mobility.

They are intended for hospital and/or home use.

These products are not suitable for any use other than that defined above.

Composition:

The bedpan is made of polypropylene with white colourant.

The male urinal is made of high-density polyethylene.

Cleaning and Disinfection:

Clean with detergent and disinfect with bleach or other disinfectants before each use.

Both the bedpan and the urinal are suitable for use in washer-disinfectors.

Service life:

The service life of the urinal and the bedpan is determined by wear and deterioration due to use.

Dispose of the products when changes in colour, roughness, deformation, etc. are detected.

Storage Conditions:

Keep away from heat sources and direct sunlight.

Symbols Glossary:



Catalogue Number



Lot Number



Expiry Date



Medical Device



Quantity



CE Marking



Keep away from sunlight



Consult the Instructions for Use at eifu.enfa.es



Manufacturer



Unique Device Identifier

In the event of a serious incident related to the product, it must be reported both to ENVASES FARMACÉUTICOS, S.A. and to the competent authority of the Member State in which the user is established.

A 'serious incident' is understood to be any incident that results in death or serious deterioration in the state of health of a patient or user, or that represents a serious threat to public health



ENVASES FARMACÉUTICOS, S.A.

C/ Paralela, 15.

28860 Paracuellos de Jarama, Madrid. España/Spain.

info@enfa.es - www.enfa.es

REVISION DATE: 2025/12

TEC176/1